

Made for  
iPhone | iPad | iPod



ReSound GN

Get a good start  
with the ReSound  
Smart 3D™ app

GN Making Life Sound Better



# Get ready to personalize your hearing experience

The ReSound Smart 3D app gives you easy, discreet, personalized control of your hearing experience.

You can use the free ReSound Smart 3D app to control and optimize your ReSound hearing instruments right from your compatible Apple® or Android™ device.

In order to prepare for your next appointment, please follow the instructions in this guide to download the ReSound Smart 3D app.

# How to download the ReSound Smart 3D app

## Download app to your Apple device

**Step 1:** Make sure your Apple device is compatible with your hearing instruments. You will find a complete list of compatible devices at [future-resound.com](http://future-resound.com).

**Step 2:** Make sure your Apple device is running the latest compatible of iOS:

1. Tap Settings > General > Software Update.
2. If a software update is available, tap Install.

Note: You must be connected to Wi-Fi to update iOS.

**Step 3:** Download the ReSound Smart 3D app from the App Store.

1. Tap the App Store icon.
2. Tap the Search icon, and then type “ReSound Smart 3D app”.
3. When the ReSound Smart 3D app appears, tap “Get.”
4. Tap “Install” and then sign into your iTunes account with your Apple ID and password to complete the download.



## **Download app to your Android device**

**Step 1:** Make sure that your mobile device is compatible with your hearing instruments. You will find a complete list of compatible devices at [future-resound.com](https://future-resound.com).

**Step 2:** Make sure that your Android mobile device is running the latest compatible software version.

**Step 3:** Download the ReSound Smart 3D app on Google Play:

1. Tap the Google Play icon.
2. Type “ReSound Smart 3D app” in the search field.
3. Tap “Install” to load the app on your phone.

## **Pairing**

You are now ready to pair your hearing instruments with your mobile device. Pairing instructions can be found in the Quick Guide for Connectivity that you receive with your hearing instruments. You can also find the Quick Guide at [future-resound.com](https://future-resound.com). If you need assistance, please contact your hearing aid center.

# My Information

Please provide the following information and bring this brochure with you to your next appointment:

My smartphone, *for example, iPhone 7:*

---

My other streaming devices, *for example, ReSound TV Streamer 2:*

---

---

---

---

---

---

---

**Manufacturer according to FDA:**    **Manufacturer according to Health Canada:**

**GN ReSound North America**

8001 E Bloomington Freeway  
Bloomington, MN 55420  
USA  
1-800-882-3636  
future-resound.com

**ReSound Canada**

303 Supertest Road  
Toronto, ON M3J 2M4  
Canada  
1-888-737-6863  
future-resound.com

© 2019 GN Hearing A/S. All rights reserved. ReSound is a trademark of GN Hearing A/S. Apple, the Apple logo, iPhone, iPad, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Android, Google Play and the Google Play logo are trademarks of Google Inc.



MK605222

MK605222 Rev A 2019.02

**ReSound GN**  
.....